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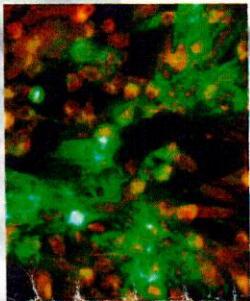
**UTM™**

## UTM™: UNIVERSAL TRANSPORT MEDIUM OFTEN IMITATED, NEVER REPLICATED.

COLLECTION | TRANSPORT | PRESERVATION

### > UTM™ ADVANTAGE

- ROOM TEMPERATURE STABLE
- RAPID RELEASE AND DISPERSION of patient sample material and virus particles during vortexing due to three glass beads in each tube.
- UNIQUE MEDIA FORMULATION includes antibiotics to inhibit bacteria and fungal flora in patient samples
- SAFE AND RELIABLE MOLDED BREAKPOINT

• SAFE, SHATTERPROOF, STAND UP tubes with distinctive internal conical shape enabling centrifugation of samples.

• AVAILABLE WITH REGULAR AND MINITIP FLOQSWABS™

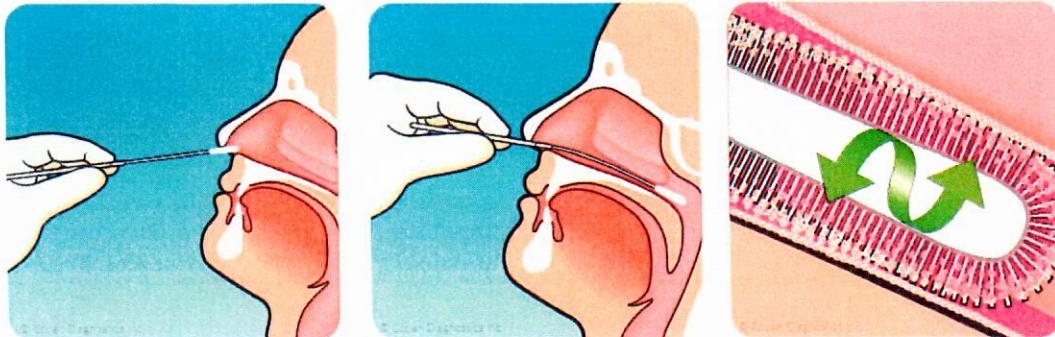
• COST EFFICIENT large 3mL media fill volume allows for multiple tests on the same specimen. 1mL fill volume prevents sample dilution.

For collection, transport,  
maintenance and long term freeze  
storage of **Virus, Chlamydia,**  
**Mycoplasma and Ureaplasma**


Available as filled tubes in bulk or as patient sample collection packs which can incorporate various combinations of filled tubes with either FLOQSwabs™ (regular or mini tip) or polyester fiber swabs.

  
**COPAN**  
Innovating Together™

## NASOPHARYNGEAL FLOCKED SWABS AND UTM™ HOW TO GUIDE

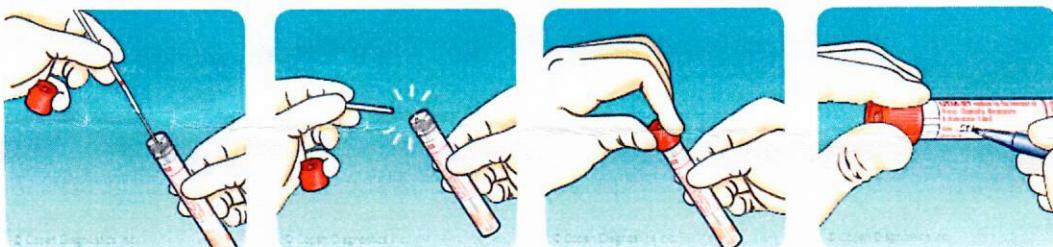- 1) Gently insert the swab along the nasal septum just above the floor of the passage to the nasopharynx until resistance is met
- 2) Rotate the swab gently against the nasopharyngeal mucosa for 10 - 15 seconds then gently remove swab

- 3) After the swab is removed from the patient place it into the tube of UTM™ transport medium all the way to the bottom of the tube
- 4) Holding the swab shaft close to the rim of the tube, break the applicator shaft at the colored breakpoint indication line. Hold the tube opening away from your face.

### ORDERING INFORMATION:

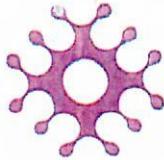
#### Individual Components:

330C      UTM™ tube for Viruses, Chlamydia,  
Mycoplasma and Ureaplasma  
503CS01    Flexible Minitip Flocked Swab

#### Collection Kit:

305C      Kit comprises UTM™ tube plus Flexible  
Minitip Flocked Swab

**Copan Diagnostics Inc.**  
26055 Jefferson Ave.  
Murrieta, California 92562 USA  
Toll Free: (800)216-4016 (US & Canada)  
Phone: (951) 696-6957 Fax: (951)600-1832  
E-Mail: [info@copanusa.com](mailto:info@copanusa.com)  
[www.copanusa.com](http://www.copanusa.com)

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